

CaptureDent

Intra-Oral Camera

INSTRUCTIONS

Read Instructions Before Use

Standard Components

1. Camera Handpiece
2. Unit Mount Handpiece Holder and screw
3. DC – 5V Adapter
4. Disposable Sleeves (100 Pieces)
5. Instruction Sheet

Set Up

Connect the power cable to the camera cable. Plug the power cable into your 110 volt outlet and the RCA (yellow) video cable into your monitor or capture system.

Attach and screw the unit mount (handpiece holder) to a suitable place or use the high speed handpiece holder of your dental unit.

If you are using a television as a monitor, turn the television to “Video”.

Operation

1. Be sure that the power supply and RCA cable are plugged in.
2. Peel off the protective film from the lens cover when using for the first time.
3. Turn on the camera by pushing the On-Off switch up. This will illuminate the six LED lights around the lens.
4. The focus is pre-adjusted for intra-oral viewing.
5. Put on a disposable sleeve over the camera handpiece for each patient.

Sterilization

The camera is not autoclavable. Always use a new disposable sleeve for each patient. Never reuse the disposable sleeve! The camera can be cleaned using a mild alcohol cloth but precautions must be taken to avoid the camera from getting wet.

Maintenance

There is no scheduled maintenance for the camera. Keep the unit dry. Avoid vibration or shock.

Specifications

Pick Up Element	¼” Color CCD
Power Supply	DC 5V
Weight	40g

Warranty

CaptureDent warrants this unit against defects in material or workmanship for a period of one year from the date of purchase. This warranty does not apply to damage arising out of abuse, alteration or improper operation of the unit. CaptureDent shall repair or replace the defective unit upon return of the unit together with evidence of purchase date to its authorized dealer.

Software

Please read the instruction manual at www.capturedent.com before installing or using the software.

CaptureDent

11357 Nuckols Road

Suite 105

Glen Allen, VA 23059

(804) 399-2552 – (804) 360-8799 (fax)